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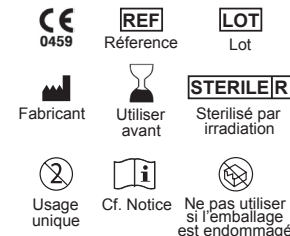
Instructions for use

CBS™ STERILE FILLING NOZZLE for CBS™ High Security embryo and sperm straws

Intended use

The CBS™ filling nozzle is designed for filling CBS™ High Security embryo and sperm straws. It guarantees an airtight seal by preventing the straw sealing area from being contaminated by the biological sample.

Cautions

- The user should read the entire Instructions for Use and understand the Cautions and Warnings and be trained in the correct procedure before using the CBS™ filling nozzle.

- Strict compliance with the instructions for use of the CBS™ filling nozzle and CBS™ High Security straws is essential to guarantee the cryogenic resistance of the straws in liquid nitrogen.

Warnings

- Manual filling of CBS™ High Security straws must be done with CBS™ filling nozzles in order to keep the sealing area clean and dry.
- When using CBS™ filling nozzles, never expel liquid back through the nozzle in order to avoid leakage between the filling nozzle and the straw and potential contamination of the sealing area.
- Only thermal seals made with a regularly maintained SYMS type sealer (Cryo Bio System, France) are guaranteed to be airtight and cryopreservation-resistant.
- Store in a clean and dry environment, away from light and heat sources.

Description

CBS™ filling nozzles are supplied as single use devices individually packaged in peel-off blisters in a protective box per 100 units. The CBS™ filling nozzle is composed of a transparent capillary overmolded with a flexible shape designed to provide an airtight connection to the open end of

CBS™ High Security straws. The capillary enters into the open end of the straw without coming into contact with the inside wall so as to keep a defined length of the straw clean and dry during the filling process and when withdrawing the filling nozzle. The small diameter and transparency of the capillary allows selective aspiration of cells from a volume under the microscope.

Filling and sealing

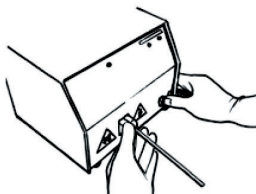
CBS™ High Security straws are filled by aspiration through a CBS™ filling nozzle. Aspiration should be done using a micro-aspirator (Ref. 014498, Cryo Bio System) or a syringe equipped with a special connector for CBS™ straws (Ref. 016730 / 016731, Cryo Bio System).

1. Insert the end of the CBS™ High Security straw that is to contain the biological sample directly and firmly into the filling nozzle. A click should be heard. The central capillary enters the straw without touching the inner surface. Check the connection by lightly pulling on the filling nozzle. It should not move.

2. Load the biological sample into the straw by applying a gentle aspiration force. Don't submerge the nozzle over the capillary to prevent the entry of liquid through the two small openings on the side. Never expel liquid back through the nozzle in order to avoid leakage between the filling nozzle and the straw and potential contamination of the sealing area.

3. After the straw has been filled, gently remove the filling nozzle by twisting it. Avoid shaking or bending the straw to prevent contact between the capillary and the inner surface. The two small openings on the side of the filling nozzle should not be blocked to enable air to enter when it is withdrawn.

4. Inspect the sealing area for contamination before sealing the end of the straw with a SYMS type sealer.
5. Disconnect the aspiration system and seal the second end of the straw as indicated above.

Disposal after use

Please follow local legal regulations for the disposal of biologically contaminated waste.

Quality assurance

Cryo Bio System is ISO 13485 certified.

Shelf-life

Three years from date of manufacture (see expiration date printed on the label).

First CE marking: 1998

0459

Reference

Batch code

Manufacturer

Use by

Sterilized by irradiation

Do not reuse

See Instructions for use

Do not use if package is damaged