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# ISOLATING CONTAINER

for the transportation of sperm prepared for I.U.I.

**Note :** refer to the guide to Good Clinical and Biological Practice, French Official Journal dated February 28, 1999.

**Article 2.1.3 :** "If sperm has to be transported, transportation should be conducted in a system enabling the system to be maintained at a temperature between 20 and 35°C and preventing temperature changes."

For the safe transportation of tubes of sperm preparations from the laboratory to the gynecologist's office, Cryo Bio System offers a new isolating container.

## Description

High-density polyethylene container filled with polyurethane foam with an internal cavity to contain the tube.

Length: 120 mm

Diameter: 65 mm

The dimensions are suitable for all 5 to 15 ml round-bottomed or conical-bottomed culture tubes available on the market.

## Use

Two hours before use, pre-heat the container to 37°C for optimum assurance of the temperature for shipment of a tube containing prepared sperm from the laboratory to the clinician for insemination. The container enables maintenance of spermatozoïd mobility while preventing thermal shocks.

## Instructions for use

Turn the lid to open or shut the container (screw off or screw on).  
Place the tube in the cavity situated in the middle of the container.  
The container is ready for use and transportation to the site where insemination is to be conducted.

If necessary, the container may be identified by affixing a label.

## Important note

The isolating container for prepared sperm tube transportation may be used with our disposable sterile intrauterine insemination device, which does not require use of a syringe (ref. 007446).

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*Cryo Bio System is part of the IMV Technologies Group.*

## Symboles Graphiques / Graphic symbols

Lot  
Lot

REF

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Date de fabrication  
Date of manufacture